## SUOMI

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## ENGLISH

### What Dermalol MEDI Moisturizing Eye Drops is and what it is intended for

Dermalol MEDI Moisturizing Eye Drops is an intensive moisturizing eye drop intended for relieving moderate and severe symptoms of dry eye. Evaporation and elevated osmolarity of the tear fluid in dry eyes damage the ocular surface, causing uncomfortable sensations in the eyes and blurring of vision. The active ingredients in Dermalol MEDI Moisturizing Eye Drops are trehalose and hyaluronan that bind water in the tear film, bring tear osmolarity back to a harmless level, and help the ocular surface recover from damage and redness. The moisturizing, protective and lubricating properties of trehalose and hyaluronan relieve the symptoms of dry eye – irritation, itching, and foreign body sensation in the eyes – caused by environmental factors such as dry air, wind, smoke, air pollution, intensive light, as well as by ocular surgery, allergies, or contact lens wear. It is the high-molecular-weight hyaluronan that stabilises the tear film on the ocular surface and offers a long-lasting moisturising effect. The composition of Dermalol MEDI Moisturizing Eye Drops promotes good vision, relieving the strain of eyes due to activities such as mobile device use, visual display work, reading, and driving. Dermalol MEDI Moisturizing Eye Drops is a sterile CE marked medical device developed and manufactured in Finland with clinically tested active ingredients. It comes without any preservatives and it is particularly suitable for sensitive eyes and for improving the comfort of contact lens wear – the lenses need not be removed from the eyes whilst using the product. The bottle does not contain materials releasing silver ions into the eye.

### Instructions for use

Tear off and discard the seal of the cap to open the bottle for the first time. Open the cap. Practise handling of the bottle before the first application. Gently squeeze the bottle in tip down position only. Squeeze the bottle with increasing force until a drop builds up at the tip and falls off. Apply 1–2 drops in both eyes 1–3 times daily. Shake the remaining drop off the tip and then close the cap. Do not put the cap on until the bottle has got its original round shape (replacement air is flowing in the bottle). A small liner pad inside the cap ensures that the next drop you take will be fresh and clean. The contents should be used within six (6) months from opening and by the expiry date.

### Precautions and warnings

- Do not use if the bottle is damaged.
- Each bottle should be used by one user only.
- Do not touch the eye or contact lens with the tip of the bottle. Do not touch the tip of the bottle with fingers.
- Do not squeeze the bottle with the cap on.
- After application, do not put the cap on until the bottle has got its original round shape (replacement air is flowing in the bottle).
- Do not use after expiry date printed on the bottle and carton box or after discard date (6 months after opening).

- If you experience blurring of vision just after application, please wait a moment before continuing activities that require clear vision. Blurring should be mild and temporary only.
- Wait at least 15 minutes between possible applications of other ophthalmic medication.
- Keep out of reach and sight of children.
- Store at 15–30 °C, protect from direct sunlight and freezing. Store under recommended conditions at all times.
- Discard the empty bottle appropriately.
- The manufacturer, Oy Finnsusp Ab, quality@piloset.fi / tel. +358 2 489110 and Finnish Medicines Agency Fimea, tel. +358 29 522 3341, must be notified of serious adverse events related to use of the product.

### Contraindications

Do not use if you are sensitive to any of the ingredients. If in doubt, consult a healthcare professional.

### Composition

2.0% trehalose, 0.25% high-molecular-weight hyaluronan in sterile isotonic, low-phosphate-buffer solution. Free from preservatives, alcohol, borates, mineral oil, or materials of animal origin.

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### Recycle packing materials

Sort box into carton waste container, leaflet into paper waste container and dispenser into plastic packaging waste container or energy waste.